

GAME READY

What is Game Ready?

- ✗ The Game Ready System provides cold and compression therapy using ice and water in a portable device. The Game Ready System is a DC-powered, software-controlled device that delivers compressed air and chilled water from the Control Unit through tubing to a Wrap that is designed for a specific body part (e.g., shoulder, elbow, knee, ankle, back) to treat pain and swelling resulting from injuries and/or surgical interventions.
- ✗ It is intended to be used by or on the order of licensed healthcare professionals in hospitals, outpatient clinics, athletic training settings, or home settings.

Indications &

Post-surgical and acute injuries to reduce edema, swelling, and pain where cold and compression are indicated.

–Game Ready



Contraindications

Compression Therapy (vasopneumatic compression) using Game Ready™ or any compression therapy device should not be used in patients:

- in acute stages of inflammatory phlebitis
- with history or risk factors for DVT or pulmonary embolus
- With significant arteriosclerosis or other vascular ischemic disease
- With a condition where increased venous or lymphatic return is not desired in the affected extremity (eg, carcinoma)
- Who have decompensated hypertonia

Cryotherapy using Game Ready™ or any cryotherapy device should not be used in patients:

- With significant vascular impairment (eg, from prior frostbite, diabetes, arteriosclerosis or ischemia)
- Who have acute paroxysmal cold hemoglobinuria or cryoglobulinemia

–Game Ready

Research

- ✗ Efficacy of the Game Ready Cooling Device on Postoperative Analgesia After Scoliosis Surgery in Children
 - First 24 postoperative hours using Game Ready resulted in decreased amount of opioid consumption (29%) and duration of hospitalization.
- ✗ Game Ready Case Studies
 - Reduced (from 30% to 50%) the time employees spend on “modified duty” after injury
 - Immediately following PRP injections to minimize pain and swelling and accelerate the healing process
 - Decrease in pain, swelling, and opioid consumption allowing a faster return to activities of daily living following surgery

Wraps

- × Variety of sizes
- × Most major body parts
 - Adjustable
 - Combo Wraps
 - Amputee



Accessories



Cost

- × Game Ready Ice Machine GRPro 2.1 Cold & Compression Therapy Unit
 - Includes control unit, connector hose, and AC adapter [redacted]
- × Game Ready Carry Bag [redacted]
- × Game Ready Dual 5' Connector Hose [redacted]
- × Game Ready Knee Wrap [redacted]
- × Game Ready Shoulder Wrap [redacted]

|

Other Brands

- × Polar Care Wave Cold Compression System
- × Breg Vpluse Cold Therapy
- × Vive Cold Therapy Machine
- × Powerplay Cold Compression Pump



ANY
QUESTIONS
?

References

- × ellon M, Michelet D, Carrara N, Vacher T, Gafsou B, Ilhareborde B, Mazda K, Ferrero E, Simon AL, Julien-Marsollier F, Dahmani S. Efficacy of the Game Ready® cooling device on postoperative analgesia after scoliosis surgery in children. Eur Spine J. 2019 Jun;28(6):1257-1264. doi: 10.1007/s00586-019-05886-6. Epub 2019 Jan 31. Erratum in: Eur Spine J. 2019 May;28(5):1254. PMID: 30847706.
- × <https://nzsportshealth.co.nz/game-ready-system/game-ready-clinical-studies/>